ENGLISH

3-IN-1 HORSE

TICK DISEASE TEST

Instruction Manual

[PRODUCT NAME]
3-in-1 Horse Tick Disease Test
For Self testing

[PACKING SPECIFICATIONS]
1 test/kit

[INTENDED USE]
Equine EHR+LYME+ANA Combined Rapid Test is a combined cassette to diagnose the presence of equine Ehrlichia antibody, equine Lyme Disease antibody and equine Anaplasma antibody in horse's whole blood.

Assay Time: 5–10 minutes
Sample: Whole blood.

[PRINCIPLE OF THE ASSAY]
Equine EHR+LYME+ANA Combined Rapid Test is based on indirect lateral flow immunochromatographic assay. The test device has three testing windows. Each testing window has an invisible T (test) zone and a C (control) zone. When sample is applied into the sample hole on the device, the liquid will laterally flow on the surface of the test strip. If there is enough Ehrlichia antibody, Lyme Disease antibody or Anaplasma antibody in the sample, a visible T band will appear in the corresponding testing window. The C band should always appear after a sample is applied, indicating a valid result. By this means, the device can accurately indicate the presence of Ehrlichia antibody, Lyme Disease antibody and Anaplasma antibody in the sample.

[KIT COMPONENTS]
1 Test Casstte
1 Plastic tube with buffer
1 Pipette
2 Lancets
1 Leaflet

[STORAGE]
The kit can be stored at room temperature (2-30°C). The test kit is stable through the expiration date (24 months) marked on the foil pouch. **DO NOT FREEZE.** Do not store the test kit in direct sunlight.

[TEST PROCEDURE]
1.Allow all materials, including specimen and test device, recover to 15–25°C before running the assay.

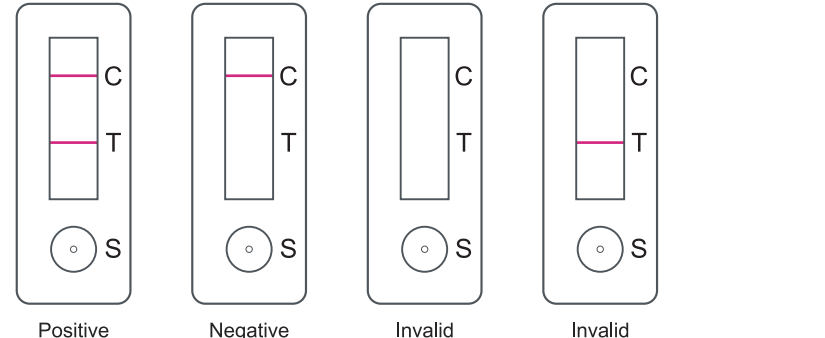
2.Take out the test card from the foil pouch and place it horizontally.

3.Use Pipette to collect horse's serum, plasma or whole blood (fresh or stored at 2–8°C for use within 3 days), drop 2 drops into a vial of assay buffer and mix well.

4.Then drop 3 drops of the diluted sample into each sample hole of the test.

5.Interpret the result in 5–10 minutes. Result after 10 minutes is considered as invalid.


[INTERPRETATION OF RESULTS]

Positive Result:
Both the C band and the T band are visible, regardless of whether the T band is strong or faint.

Negative Result:
Only a clear C band is visible.

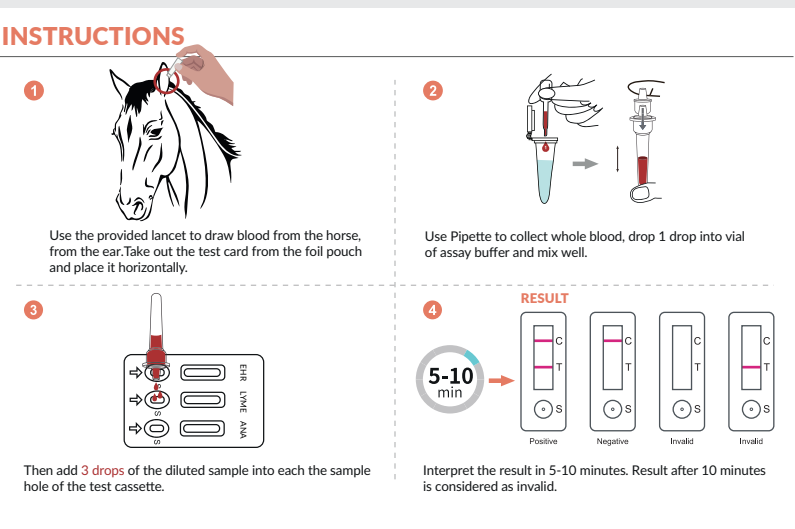Invalid Result:
No colored C band appears, regardless of whether the T band is visible. A retest is recommended.

Note: Please read the results within the specified time: Interpreting less than or exceeding time may lead to erroneous results.

[PRECAUTIONS]
- For best results, please strictly adhere to these instructions.
- All reagents must be at room temperature before running the assay.
- Do not remove test cassette from its pouch until immediately before use.
- Do not reuse the test kit.
- Do not use the test beyond its expiration date marked on the foil pouch.
- The components in this kit have been quality control tested as standard batch unit. Do not mix components from different lot numbers.

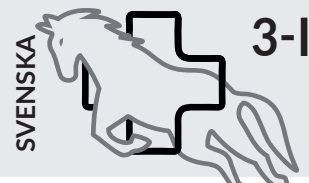
[LIMITATIONS]
Equine EHR+LYME+ANA Combined Rapid Test is designed as a simple self-test for horses. If the result is positive, you should promptly contact your veterinarian for further examination and cnfirmation. Test results should always be evaluated together with clinical signs and professional veterinary advice. For the most accurate confirmation, your veterinarian may recommend additional methods such as IFA.

Supplier: Guangzhou Herun Biopharm Co., Ltd.
Adress: No.18, Shaobai Rd., Zengcheng District, Guangzhou City, P.R. Kina
Edition: 2025/02/28

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Produsent: Guangzhou Herun Biopharm Co., Ltd.